



USER GUIDE



MALE CATHETERIZATION TRAINER

This product is available
in light and dark skin tones

60850

60870

FOR MORE SKILLS TRAINING PRODUCTS VISIT

limbsandthings.com

Sussex Street, St Philips, Bristol, BS2 0RA, UK | sales@limbsandthings.com | +44 (0) 117 311 0500

PACKAGE SUPPLIED	3
SAFETY & CLEANING INSTRUCTIONS	4
SET UP	5
• PLACING PELVIC SHELL	5
• FITTING THE FORESKIN	6
• ASSEMBLING THE WATER SYSTEM	7
• EMPTYING WATER	9
• REASSEMBLY	10
AUGMENTED REALITY TRAINING	12
• SAFETY & CLEANING INSTRUCTIONS	12
• AUGMENTED REALITY SET UP	13

TAKE A LOOK
AT OUR **VIDEO**



VISIT THE WEBSITE
limbsandthings.com

OR YOUTUBE CHANNEL
youtube.com/limbsandthings

PACKAGE SUPPLIED

Male Catheterization Module

60855

60874



Water System


60893



Pelvic Shell & Stand

60864


60881

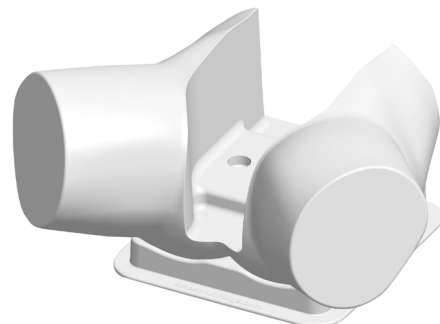

Catheterization Foreskins (x3)

60859

60878

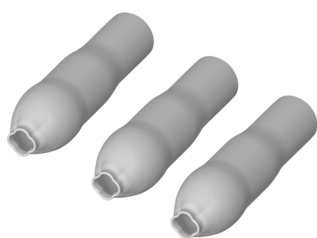

Aseptic Catheterization Sleeves (x2)

60862




Lubricant Gel with Syringe

60892



Antifungal Agent (Milton)

120-156

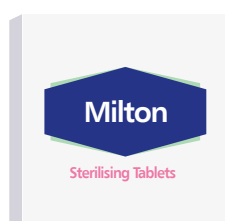

Plastic Kidney Dish




Lubricant Gel Sachet

027-113


Lubricant Gel

Water Extension Tubing

60867

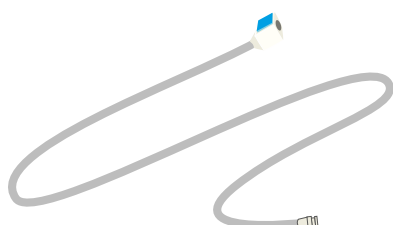


14 French Foley Catheter

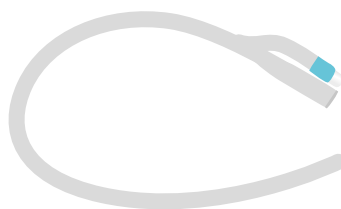

Augmented Reality Mat

60131

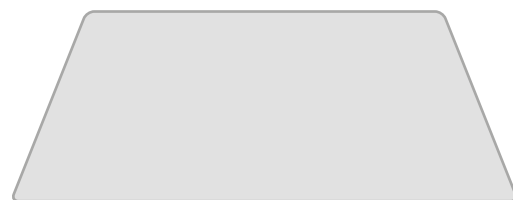

SAFETY & CLEANING INSTRUCTIONS

READ THE SAFETY INSTRUCTIONS CAREFULLY BEFORE USING YOUR CATHETERIZATION TRAINER

DO's



Follow all instructions.
Keep these instructions.
Pay attention to all warnings.

Only use attachments/accessories specified by Limbs & Things.



Refer all servicing to qualified service personnel. Servicing is required when the apparatus has been damaged.

Wear gloves at all times and remove all jewellery items. Sharp surfaces such as finger nails and rings can lead to damage of silicone parts.



Use lubrication when inserting catheter.

Use only the Milton Antifungal agent supplied to sterilize water.



Use 14 French Foley catheter (male length).

Drain all water from the product and clean with a damp, soft cloth or sponge, using only warm water with mild detergent.



DON'Ts



Do not install near any heat sources, expose the product to flames or use near naked flame sources.
Do not use flammable solvents near or on this product.

Do not set up the product near electrical appliances or items damaged by water.



Do not leave catheters inserted in the Trainer.

Do not attempt to remove a catheter which has an inflated balloon.



Do not use anaesthetic gel – use mock gel supplied with trainer.

Do not disassemble the product or attempt to repair a faulty unit. Please return the product to Limbs & Things for assessment and repair.




Do not leave trainer set up with water in. Disassemble & clean after every use.

STORAGE & TRANSPORTATION




Always ensure that the product is properly packed and secured during transportation and storage in order to prevent personal injury or damage to the product.


1

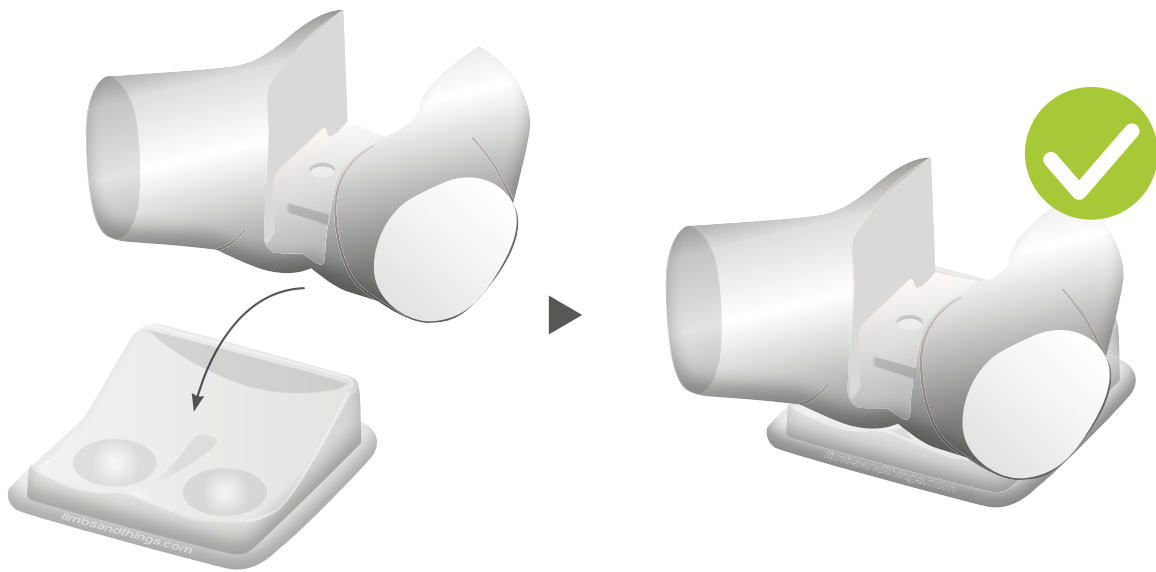


2

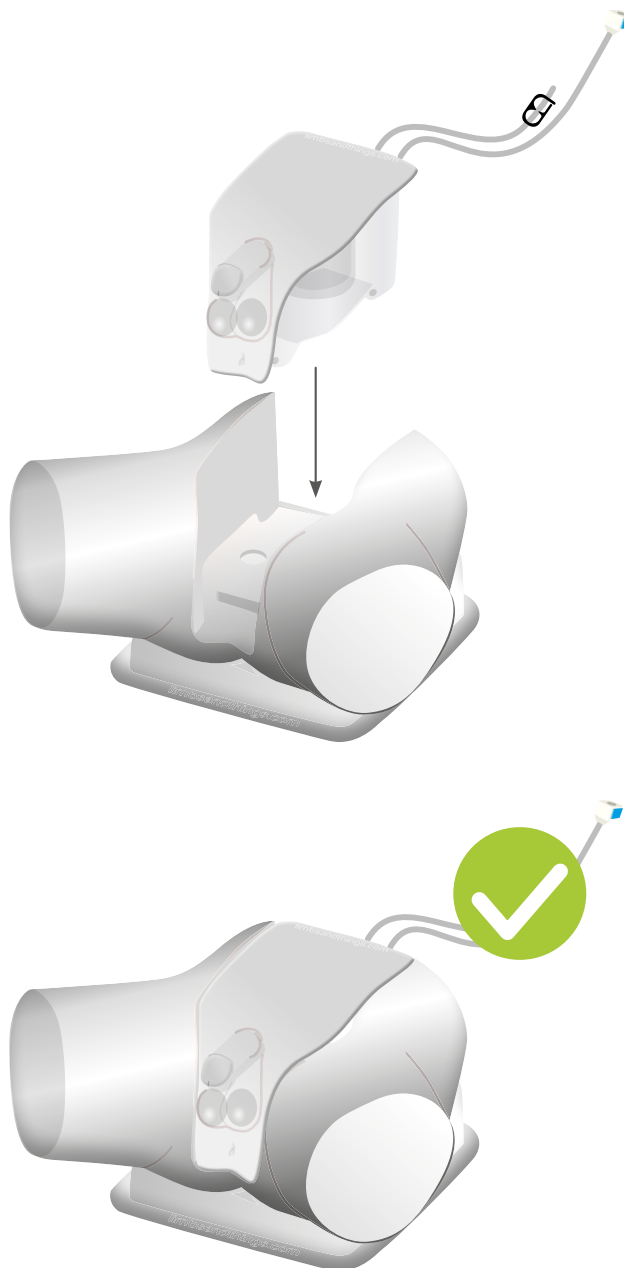


1

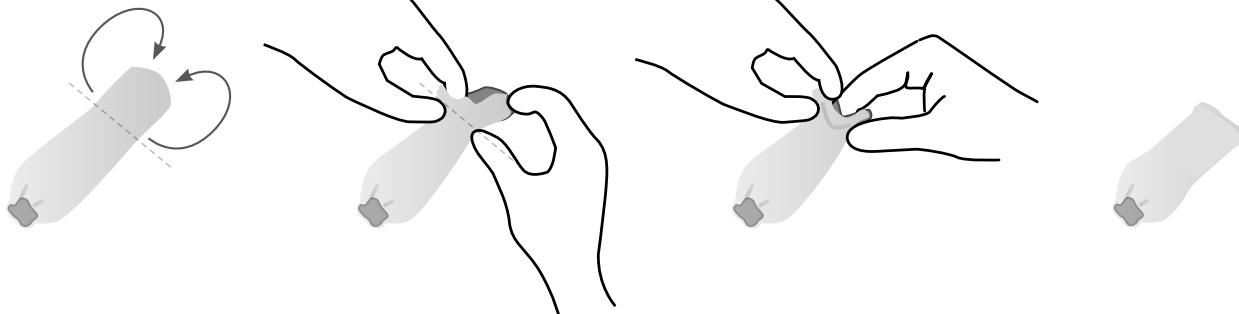

2




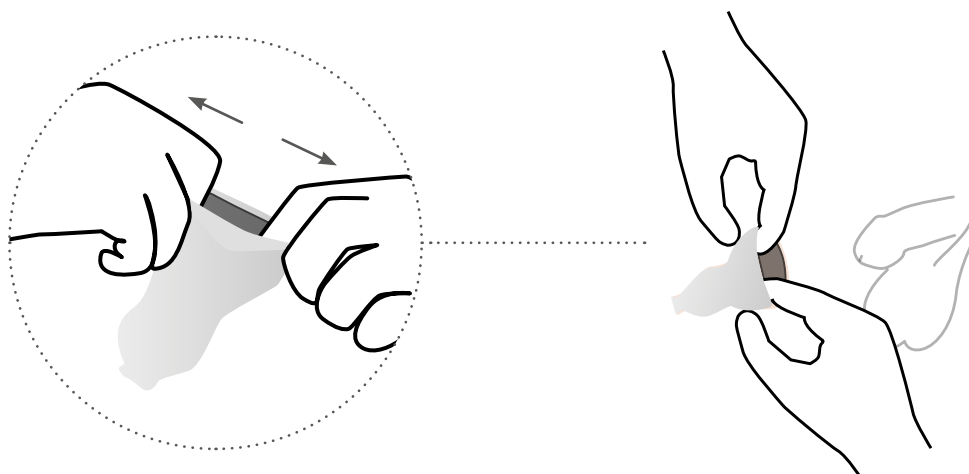


3

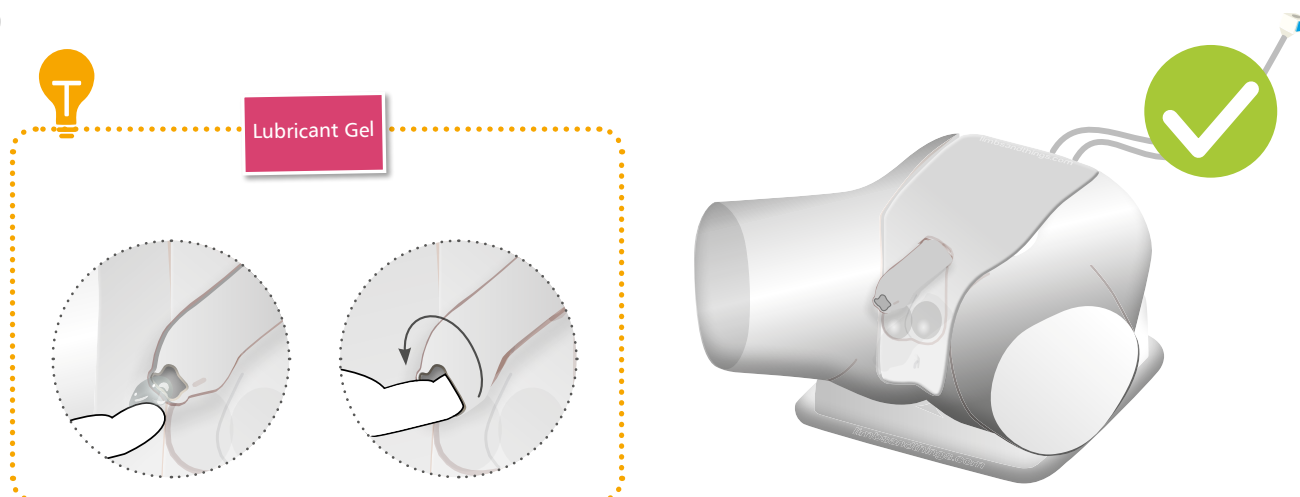


SET UP


ASSEMBLING THE WATER SYSTEM

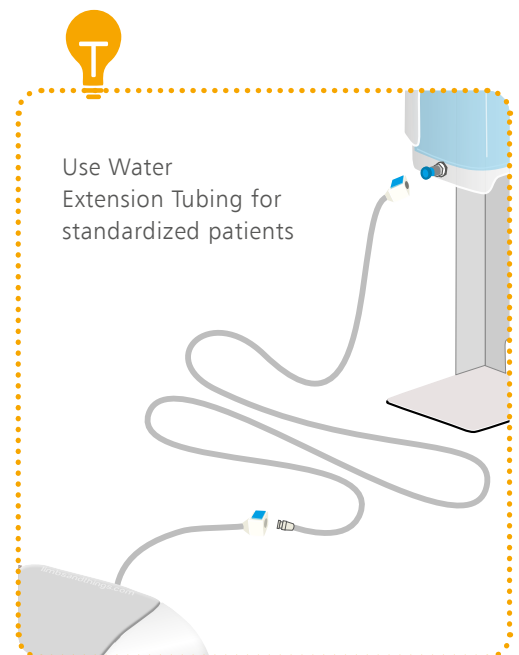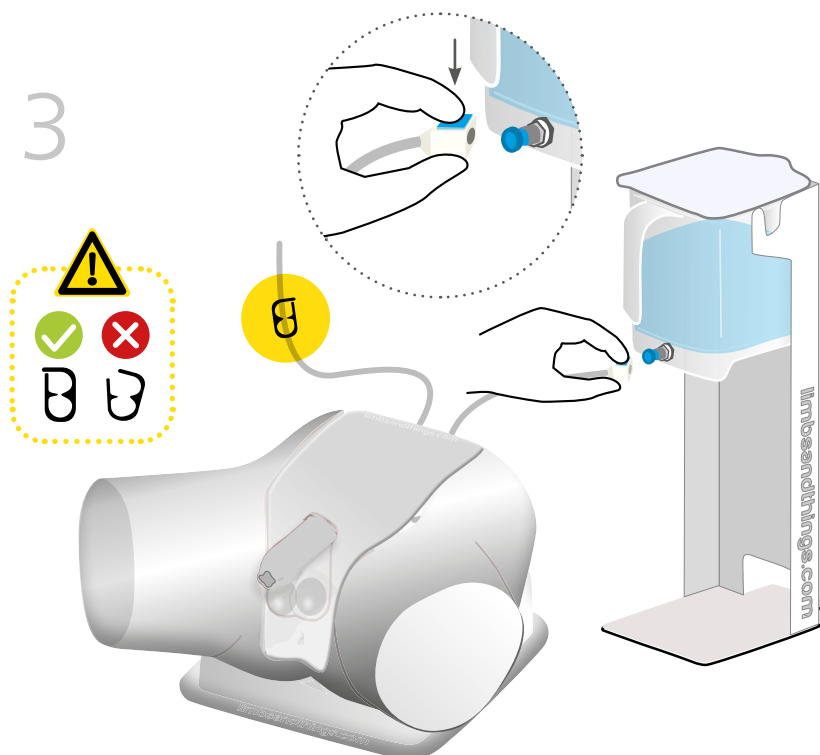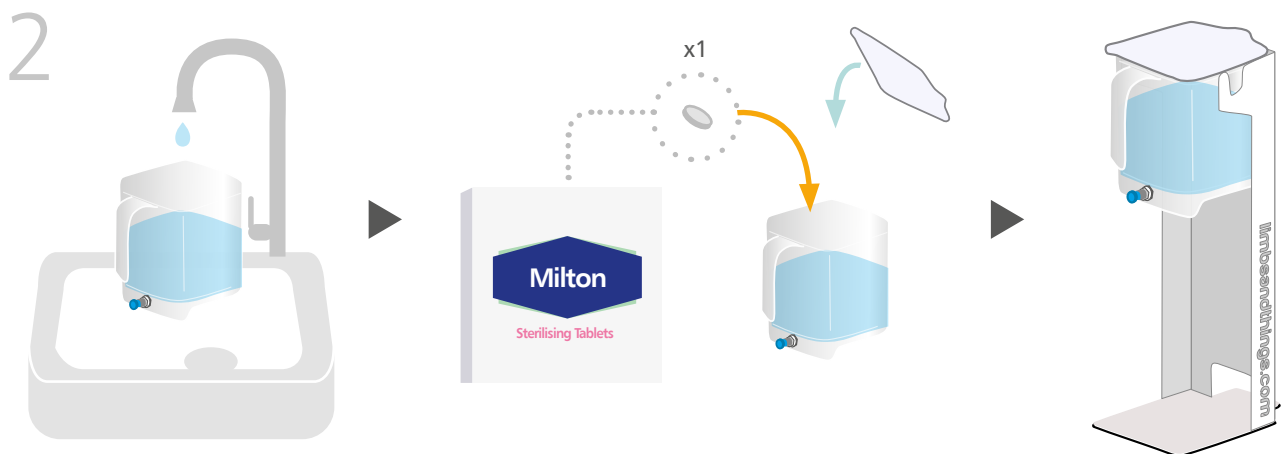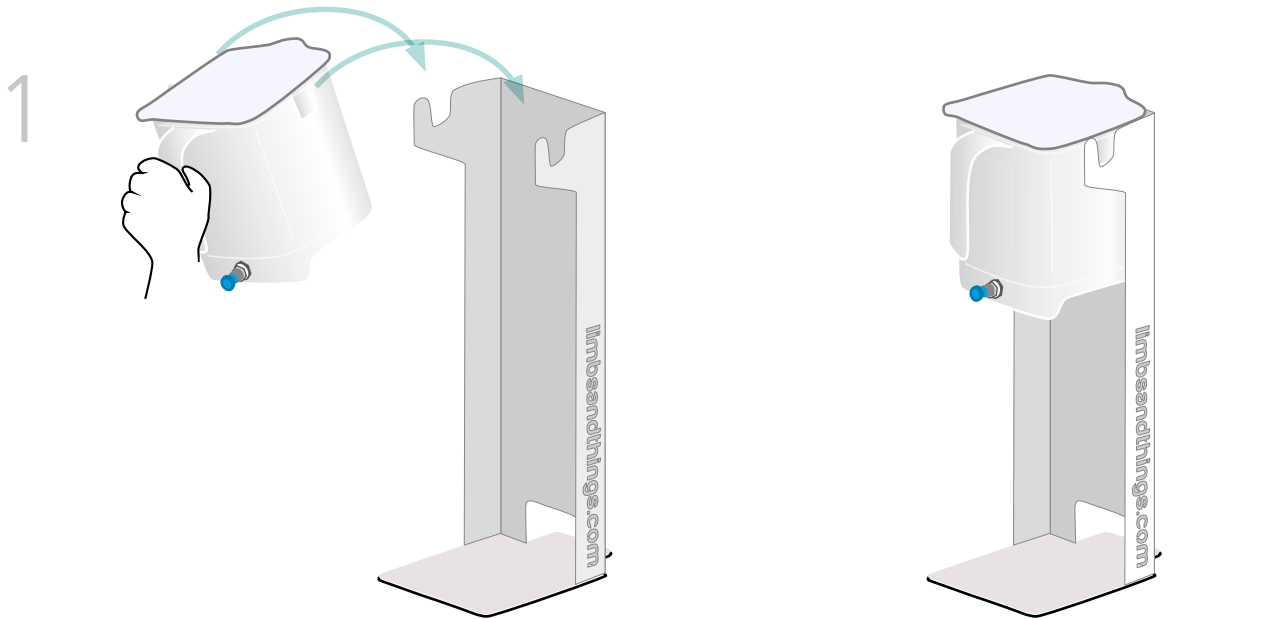

4

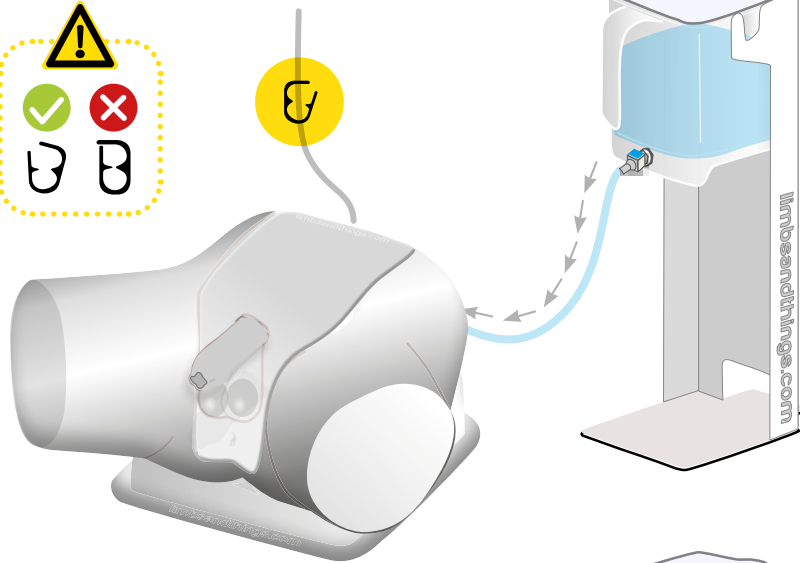


5

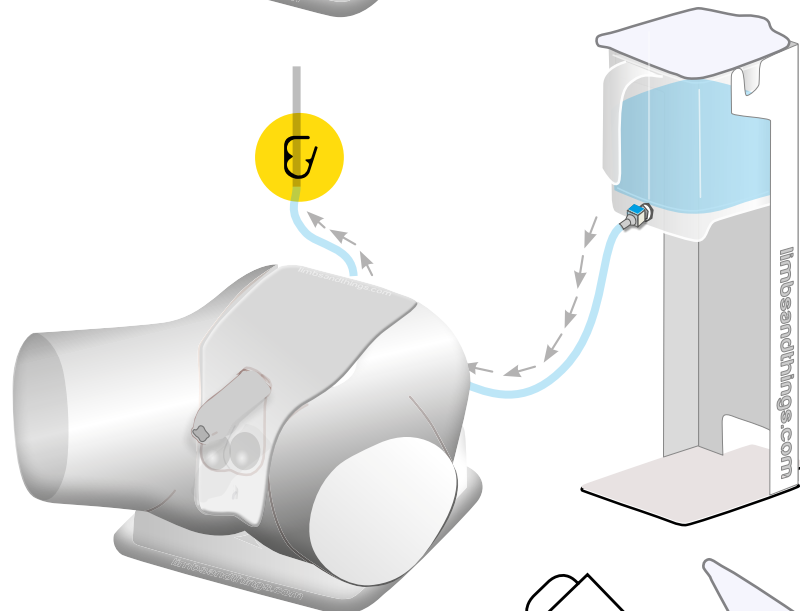

6



10ml during set up.
5-10ml per catheter
insertion

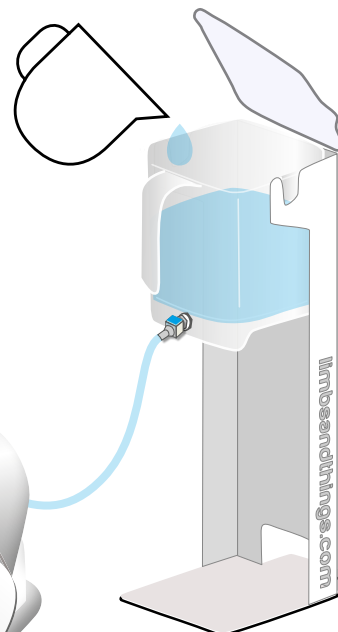


Keep water
reservoir topped up




SET UP

EMPTYING WATER


1

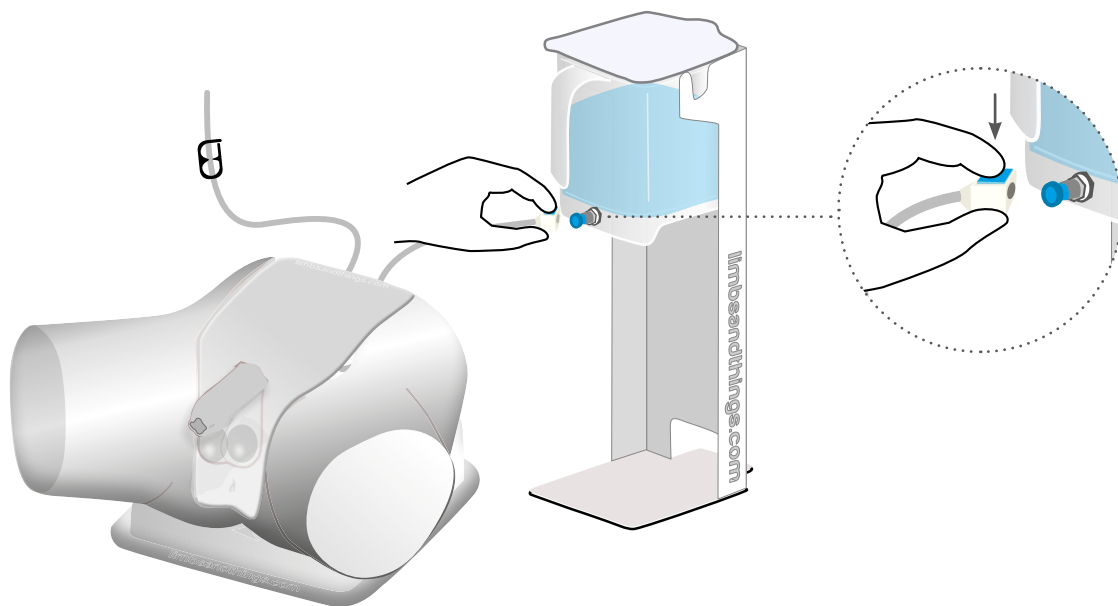

2

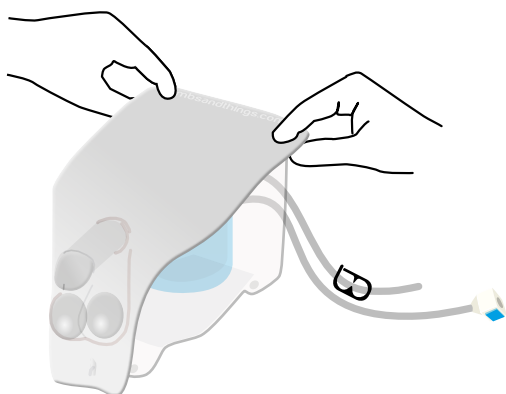


3

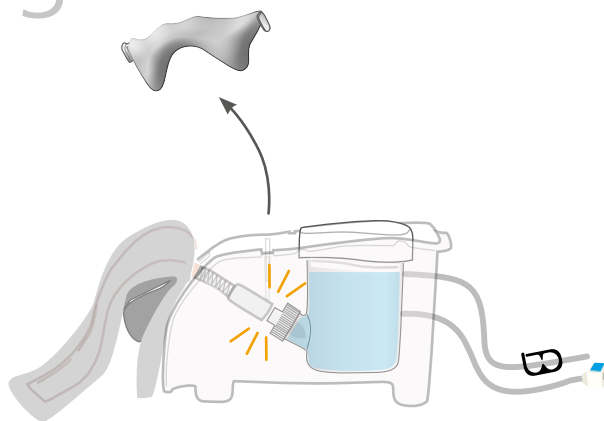



4




Dry all parts with paper towel
Reassemble and put in box

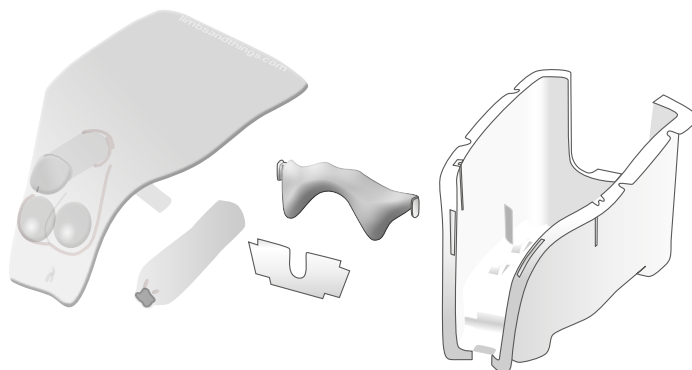


1

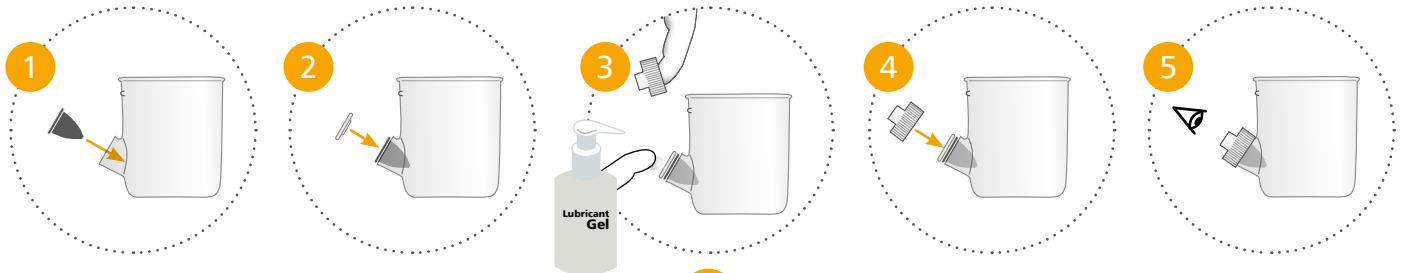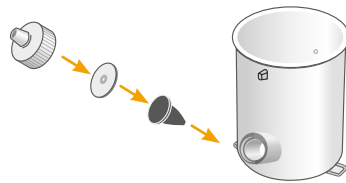



Add lubrication to diaphragm
before replacing lid

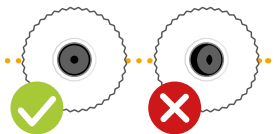


2

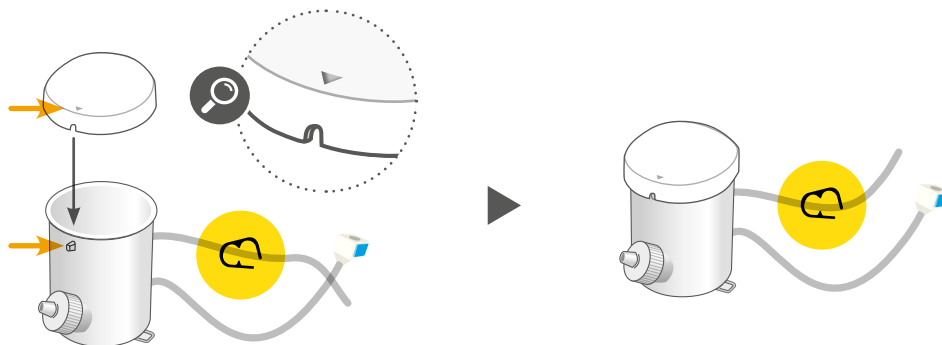


3

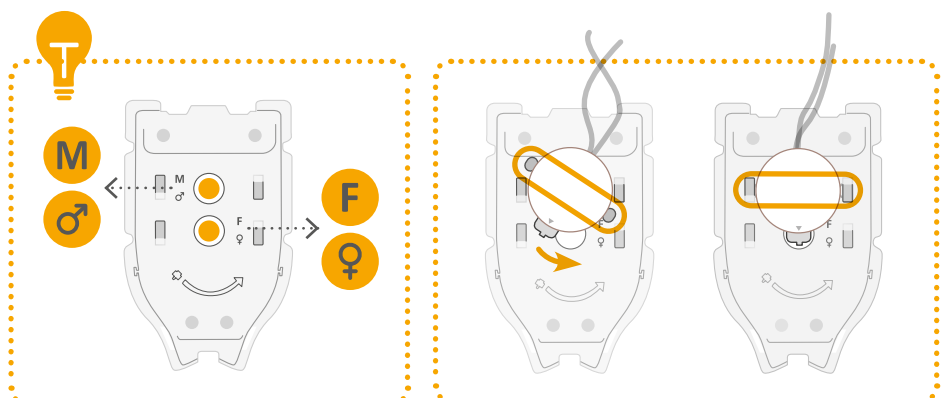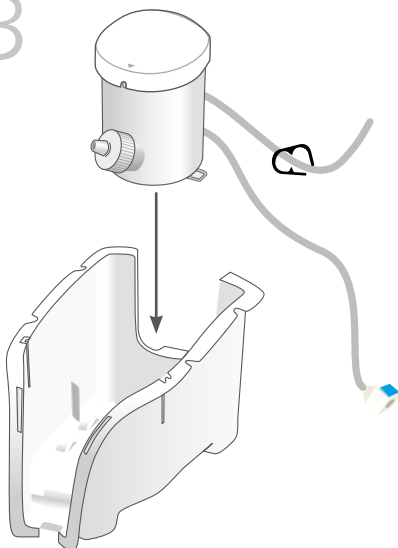


4

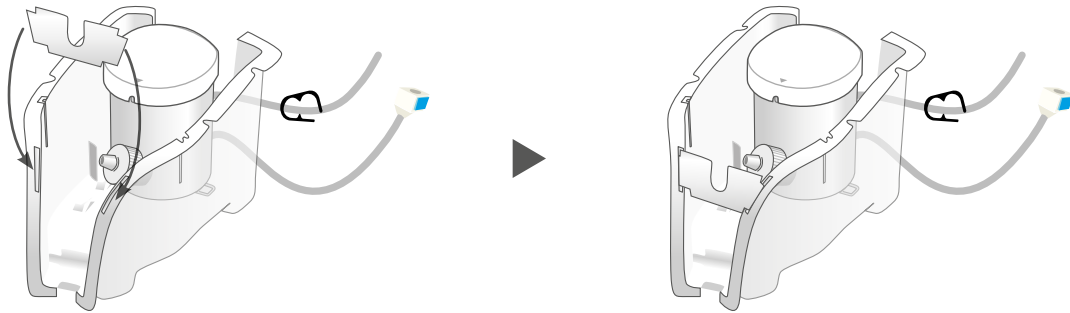


5

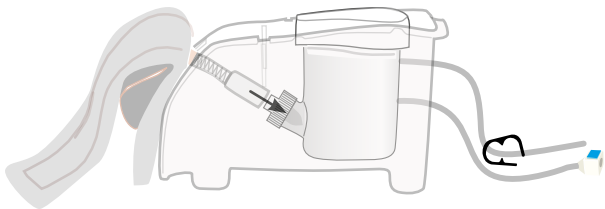

6

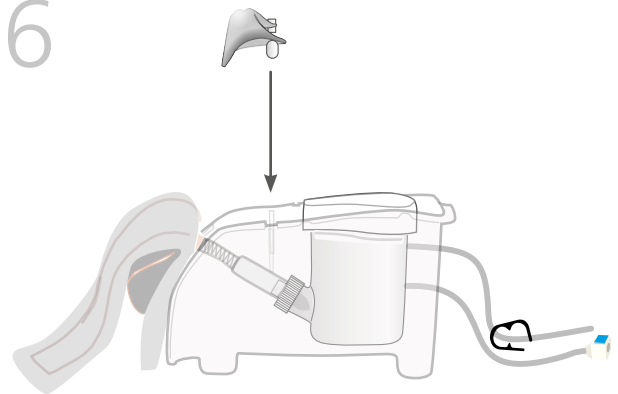

7

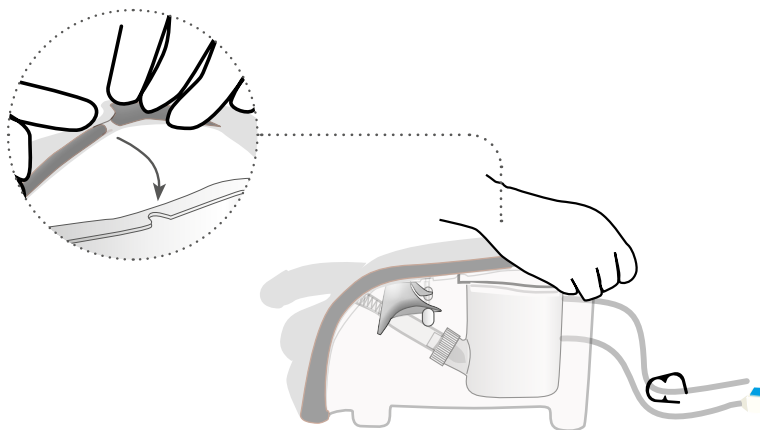



AUGMENTED REALITY TRAINING

for your Catheterization Trainer

SAFETY & CLEANING INSTRUCTIONS

Read the safety instructions
carefully before using your
Augmented Reality Mat

DO's



Follow all instructions.
Keep these instructions.
Pay attention to all warnings

Use mat on flat surface



Only use attachments/accessories
specified by Limbs & Things

Refer all servicing to qualified service
personnel. Servicing is required when
the apparatus has been damaged



DON'Ts



Do not install near any heat sources,
the product to flames or use near
naked flame sources.
Do not use flammable solvents near
or on this product

Do not set up the product near
electrical appliances or items
damaged by water



Do not cover the graphic of the mat
with other items

Do not move the mat with a
model on top of it



STORAGE



Store rolled up with print on the
outside



Don't fold



Don't place heavy objects on top

CLEANING



Can be cleaned with water, soap, and
non-alcoholic cleaning agents



Avoid abrasive and
alcoholic cleaning agents




Make sure mat is dry before storing

AUGMENTED REALITY SET UP


1

Take the mat out of tube

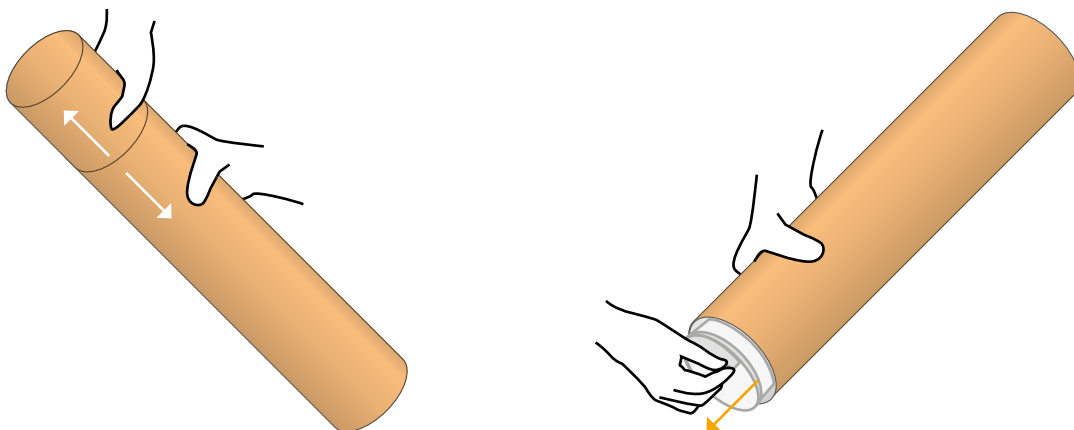

2




Place the mat on a table

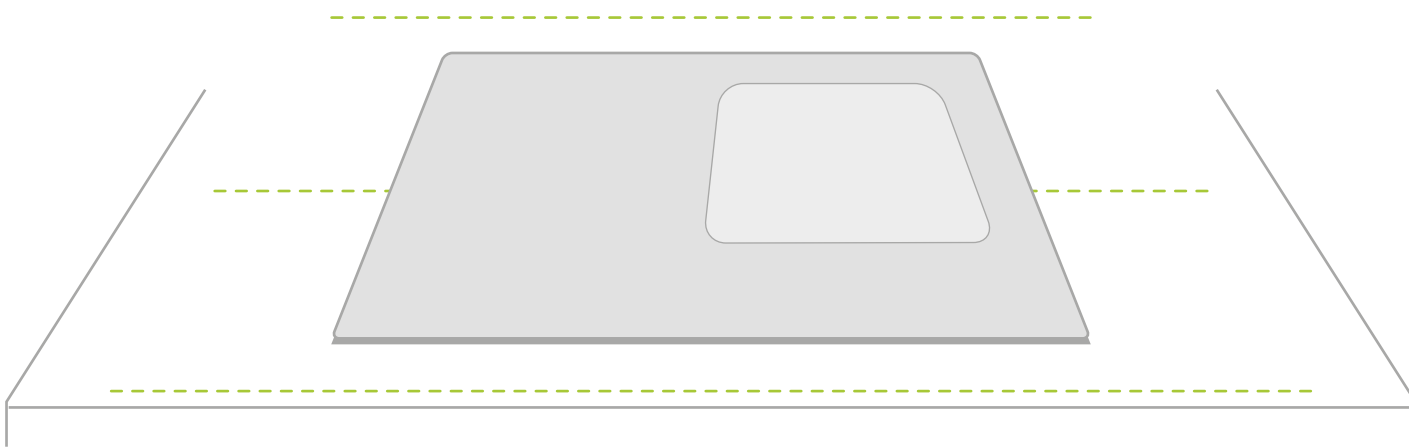


Note: Diagrams are for illustrative purposes, your product may differ from the example shown

3

Place the model on the designated area of the mat

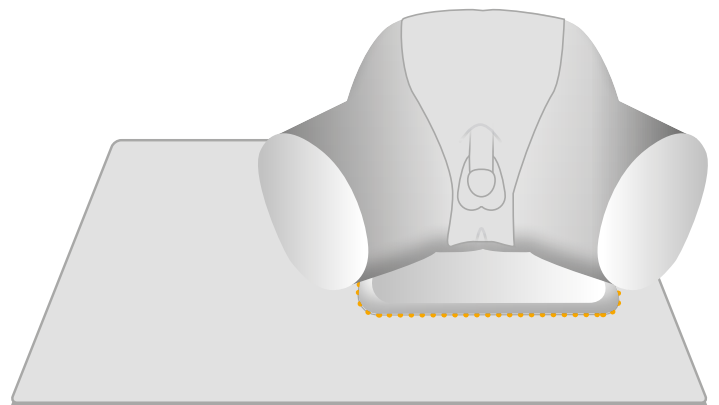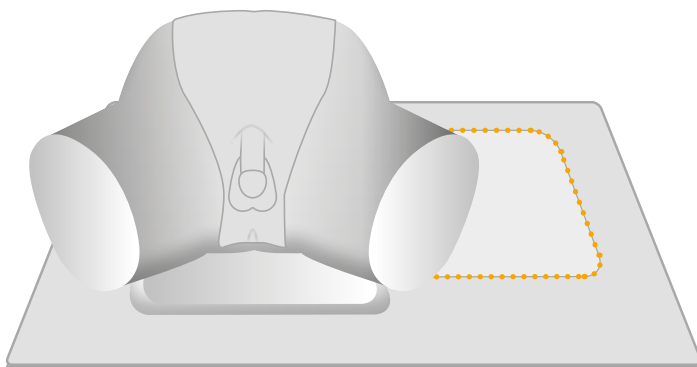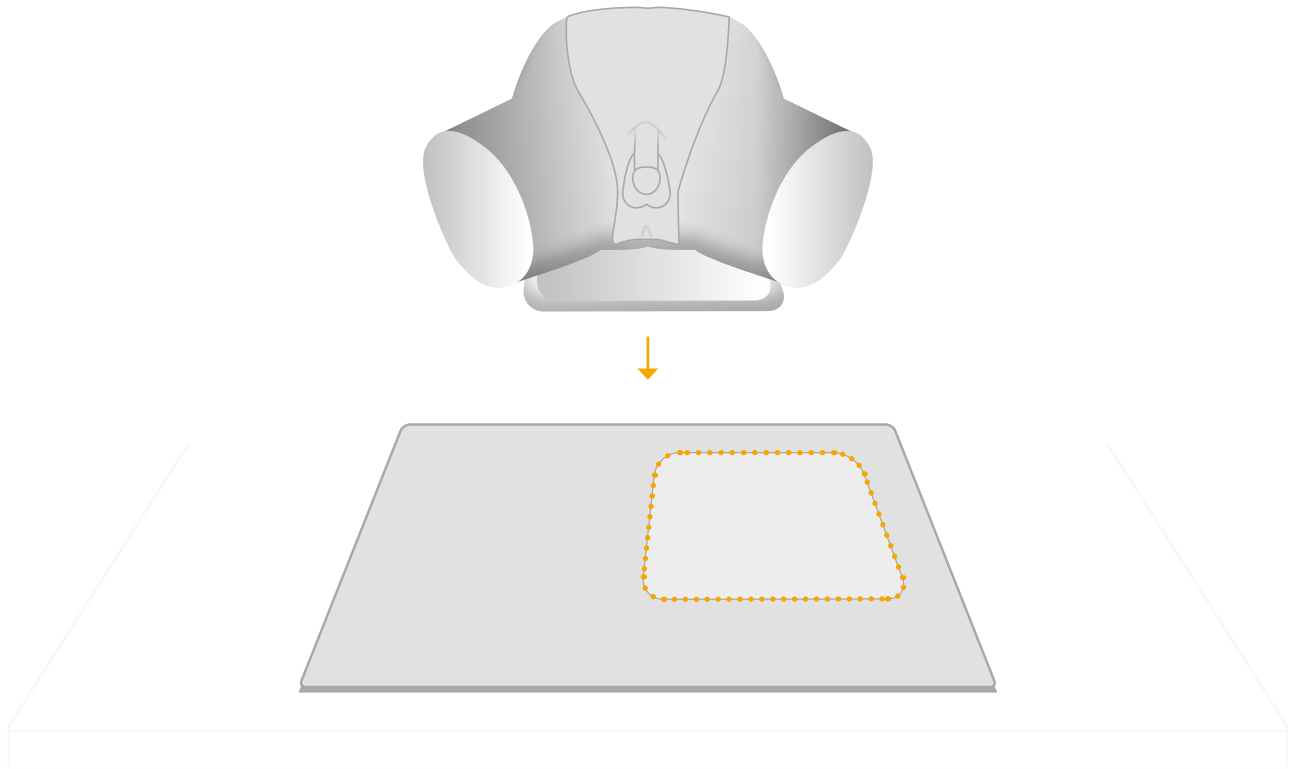

Note: Diagrams are for illustrative purposes, your product may differ from the example shown

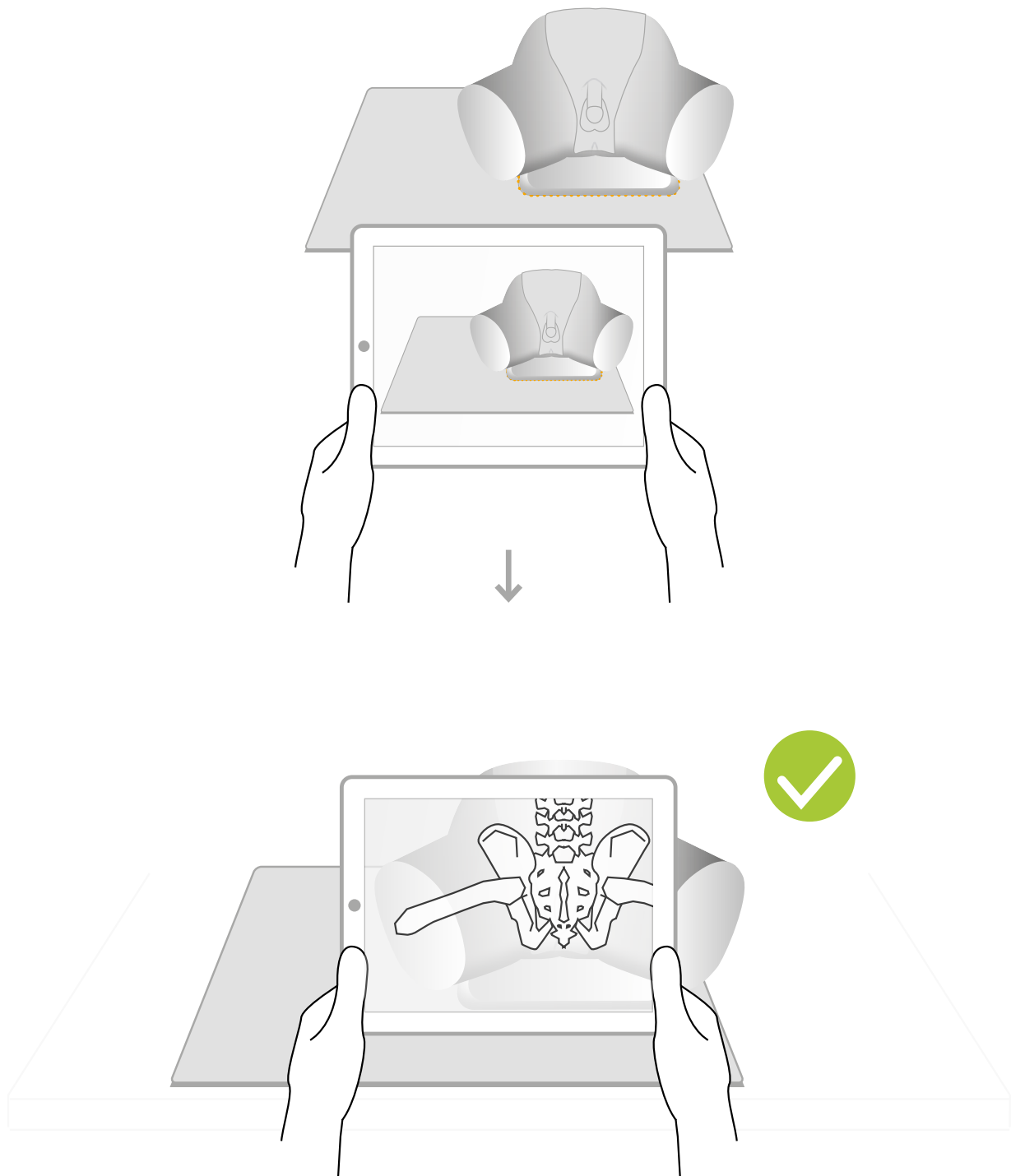

For app troubleshooting please visit the website.


Note: Diagrams are for illustrative purposes, your product may differ from the example shown

YOU MAY
ALSO BE
INTERESTED IN...

FEMALE
CATHETERIZATION
TRAINER

60851


60869

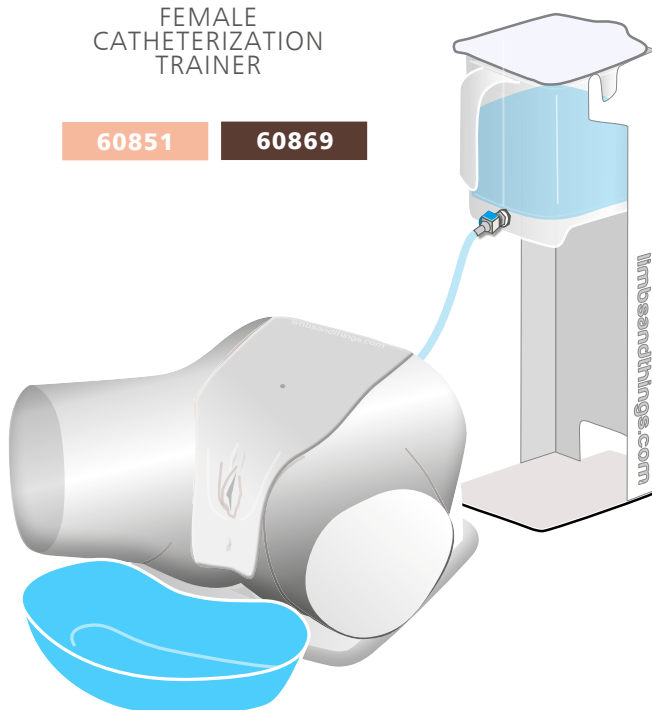

SUPRAPUBIC
INSERTION UNIT

60852

60873

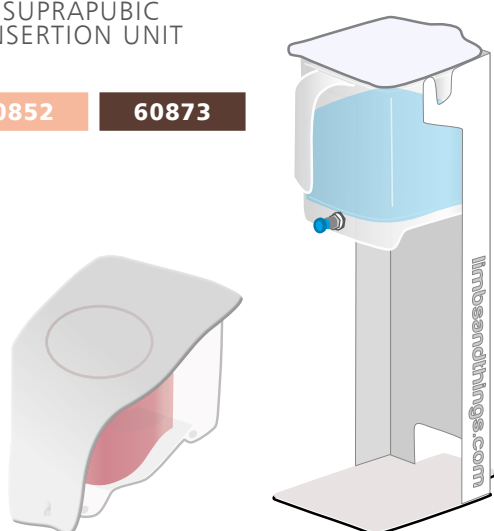

MEDIUM
CARRY CASE

40209



limbs&
things

Part No: 965-508

Issue 08, February 2026
© 2026 Limbs & Things



FM 818255